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Product information and other clinical cases:
www.vt-cube.com

INSTRUMENTARIUM

INSTRUMENTARIUM DENTAL develops, manufactures and markets high-tech systems and solutions for dental and maxillo-facial imaging. We work in close co-operation with dental professionals, universities and other research centers in our quest to develop solutions that will meet and exceed the expectations of our customers. As the establisher of panoramic x-ray imaging we are committed to providing high clinical performance while still maintaining simplicity, ease of use and workflow efficiency. The Instrumentarium Dental product portfolio consists of premium quality imaging solutions:

- The 200-generation of the legendary Orthopantomograph® x-ray family, which serves demanding panoramic and cephalometric diagnostic needs both in film and digital environments.
- VT option for digital Orthopantomograph® units – a Volumetric Tomography imaging tool for implant site evaluation.
- Focus™ x-ray and Sigma™ M direct digital sensors – an intelligent solution for advanced intraoral imaging.

Instrumentarium Dental reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your Instrumentarium Dental Representative for the most current information. CE marked according to Medical Device Directive (MDD) 93/42/EEC. Electrical safety according to IEC 60601-1. Operations comply with ISO 13485:2003, ISO 9001:2000, and ISO 14001:2004.

Headquarters

Instrumentarium Dental
P.O.Box 20
FI-04301 Tuusula, Finland
Tel. +358 45 7882 2000
Fax +358 9 851 4048

Americas

Instrumentarium Dental INC.
300 West Edgerton Avenue
Milwaukee, Wisconsin
53207 U.S.A
Tel. +1 414 747 1030, 800 558 6120
Fax +1 414 481 8665

Germany

Instrumentarium Dental GmbH
Schutterstrasse 12
77746 Schutterwald, Germany
Tel. +49 781 28 41 980
Fax +49 781 28 41 9830

France

Instrumentarium Dental S.A.R.L.
P.A. des Petits Carreaux
12 Avenue des Coquelicots
94385 BONNEUIL
sur MARNE Cedex, France
Tel. +33 1 41 94 16 10
Fax +33 1 43 77 24 90

Italy

Instrumentarium Dental S.R.L.
Via Forlanini 71,
20033 Desio (Mi), Italy
Tel. +39 0362 331 191
Fax +39 0362 300 067

www.instrumentariumdental.com